SureTact™ G3

Sectional Matrix System

User instructions (*ENGLISH*)

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SureTact™ G3

User instructions

The SureTact G3™ Sectional Matrix System is designed to create tight, anatomical contacts on Class II composite restorations. The contoured sectional matrix bands create proper tooth anatomy while three styles of rings create the required tooth separation and band adaptation for a tight, natural contact.

 Non-sterile.  Rx only  Always use a rubber dam.

Important Cleaning and Sterilization Instructions:

SureTact G3™ Rings: Remove all visible debris after use and prior to sterilization. DO NOT use solutions containing acetone or ketone. Steam autoclave sterilization only.

SureTact G3™ Sectional Matrix Bands: Matrix bands are single use disposable items. Sterilization prior to use is recommended and can be performed using steam autoclave.

Instruments & Forceps: Instruments may be dry heat sterilized, steam autoclaved or cold sterilized.

SureTact G3™ Wedges: Wedges are single use, disposable items. Spray disinfect only. Do not autoclave.

For complete sterilization instructions go to:
www.marksondental.com/sterilization

Technique for Class II restorations utilizing SureTact G3™ Rings

- 1. Place rubber dam.** Isolation of the area helps to prevent contamination from blood and saliva. As with many dental procedures, aspiration of small parts is a valid concern.
- 2. Place WedgeShield®** prior to preparation to pre-separate and protect the adjacent tooth. (figure 1) Prepare tooth with conservative Class II preparation. Once the preparation is complete, remove the WedgeShield® and place your matrix.
- 3. Place SureTact G3™ sectional matrix band.** Select a matrix band that most closely

figure 1

approximates the occlusogingival height of the tooth. The Grab-Tab should be oriented toward the occlusal margin and may be bent over the adjacent tooth after placement. (figure 2)

- 4. Insert SureTact G3™ wedge.** Keep a finger on the matrix band to prevent the wedge from dislodging the band during insertion. Wedge firmly, a high viscosity resin packed into the preparation may displace the band away from the tooth. (figure 3)
- 5. Apply the SureTact G3™ ring retainer.** Select a retainer ring that is appropriate for the embrasure and tooth being restored. Spread it with the SureTact G3 Ring Placement forceps and place it over the wedge. (figure 4)
- 6. Very Important:** push the matrix band firmly into contact with the adjacent tooth in the desired contact area.
- 7. Use your preferred etching, disinfecting and bonding technique** following manufacturer's instructions. Apply your preferred composite resin following manufacturer's instructions.
- 8. Remove the ring, wedge and band.**
- 9. Contour and polish the restoration.**

figure 2

figure 3

figure 4

Stacking of Rings & MOD Restorations

The SureTact G3™ Rings are designed to allow stacking of rings for performing M.O.D.s and multiple-tooth restorations. Rings may be stacked over each other. (figure 5)

figure 5

*(Markson SureTact G3™ Ring Placement Forceps will hold the rings more securely and provide ample force to open the rings. NOTE: Most rubber dam clamp forceps will not open the SureTact G3™ Rings far enough to allow placement.)

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© All rights reserved.
ONVET Medical Canada Inc.